Dino-Lite

Digital Microscope **Medical**

Medical Manual

www.dino-lite.eu/medical

English	1
Français	20
Deutsch	41
Español	61
Italiano	81
Nederlands	101
Svenska	121
Polski	140
Dansk	159
Hrvatski	179
Português	199
Suomi	216
Русский	234

System requirements / Exigences du système /
Systemanforderungen / Requisitos del sistema /
Requisiti di sistema / Systeemvereisten /
Wymagania systemowe / Systemkrav / Systemkrav
- Windows 10, 8, 7 (SP3), MAC OS 10.9 or above /ou plus
/ oder höher, o posterior / o superior /
of hoger/ lub nowsze / eller højre/ eller over /lub
wyzszy / eller ovanför / eller over
- USB 2.0 port
- 4 GB RAM
- 1GB graphics card / memoire graphique /
Grafikspeicher / memoria gráfica / memoria grafica /
grafisch geheugen / pamieci graficznej / grafikminne /
grafisk hukommelse / karta graficzna / grafikkort /
grafikkort
- 10 GB free disk space / espace de disque libre / freier
Festplattenspeicher / espacio libre en disco / spazio
libero su disco / vrije harde schijfruimte/ wolnej
przestrzeni na dysku / ledigt hårddiskutrymme
/ fri harddiskplads / wolne miejsce na dysku /
ledigt diskutrymme / ledig diskplads
- Motion-JPEG (MJPEG) Codec: may be required / peut
être nécessaire / kann erforderlich sein / puede ser
necesario / potrebbe essere necessario / kan
noodzakelijk zijn/ moze byc wymagane /
kan krävas / kan kræve / moze byc wymagane / kan
krävas / kan være påkrævet /

ENGLISH

Thank you for purchasing a Dino-Lite microscope. The DinoCapture software is designed to give you the best possible digital microscopy experience by the inventors of the handheld digital microscope. The DinoCapture software runs on computers with a Windows 7/8/10 operating system. The DinoXcope software is designed to work with Apple Macintosh OS.

Important safety information

- Avoid touching the lens with finger to protect the product from electrostatic damage
- Do not drop
- Keep dry

Medical device Class 1 - According to Medical Devices Regulation (EU) 2017/745

Copyright 2021, IDCP BV. All rights are reserved, nothing from this publication, neither whole or part, may be copied, duplicated, reproduced or distributed without the express written authorization from IDCP BV.

Instructions for Use: P/N M1, rev. 5, 2021Q1

Consult Instructions before Use

Intended Use

Please check the label on your Dino-Lite to see what model you have and find the appropriate page about the intended use in this manual.

IDCP BV
Manuscriptstraat 12-14,
1321NN Almere, The Netherlands
Tel. +31206186322
Fax +31 20 6189692
E-mail: info@dino-lite.eu
Web: www.dino-lite.eu/medical

Index

Product overview	3
Software installation	7
Software interface DinoCapture / Windows	9
Software interface DinoXcope / Mac	15
Cleaning and maintenance, warranty, support	16

Package contents

1. Dino-Lite digital microscope
2. DinoCapture and DinoXcope software on CD
3. Protective pouch
4. Instructions for use
5. Model-specific accessories (see table below)

Product overview

Please make sure that you have the right Dino-Lite model for your application. You can find the product designation on the label attached to the cable.

Dino-Lite Medical User Manual

Model	Description	Resolution	Magnification
MEDL7DW	DermaScope Polarizer HR	5.0 MP	10-50x
MEDL4DW	DermaScope Polarizer	1.3 MP	10-50x
MEDL7DM	DermaScope Polarizer HR 200x	5.0 MP	-10-70 & 200x
MEDL4DM	DermaScope Polarizer 200x	1.3 MP	-10-70 & 200x
MEDL4E	EarScope Pro	1.3 MP	55-90x
MEDL4EP	EarScope Pneumatic	1.3 MP	55-90x
MEDL3E	EarScope Basic	0.3 MP	20-30x
MEDL4N Pro	CapillaryScope 200 Pro	1.3 MP	20-200
MEDL4N5 Pro	CapillaryScope 500 Pro	1.3 MP	-500x
MEDL4N5	CapillaryScope 500	1.3 MP	-500x
MEDL7HM	TrichoScope Polarizer HR	5.0 MP	-10-70 & 200x
MEDL4HM	TrichoScope Polarizer	1.3 MP	-10-70 & 200x
MEDL3H	TrichoScope Basic	0.3 MP	-10-70 & 200x
MEDL4R	IriScope	1.3 MP	10-20x
MEDL4FW	PodoScope	1.3 MP	10-50x

USB 2.0	LED's	Polarizer	Metal Housing	specific accessories
yes	8 white	yes	yes	open cap, long cap, closed contact cap
yes	8 white	yes	yes	open cap, long cap, closed contact cap
yes	8 white	yes		closed contact cap
yes	8 white	yes		closed contact cap
yes	6 white	no		specula (3-4-5 mm)
yes	6 white	no		specula (2,5-4 mm)
yes	4 white	no		specula (3-4-5 mm)
yes	8 white	yes		nailshaped cap
yes	8 white	yes		nailshaped cap
yes	8 white	no		
yes	8 white	yes	yes	closed contact cap
yes	8 white	yes		closed contact cap
yes	8 white	no		
yes	2 white	no		
yes	2 yellow 8 white	yes		open cap, long cap, closed contact cap

DermaScope models

Intended Use

The DermaScope is intended to make images of spots of the intact skin as part of a dermatological diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. This model has a polarization function that can be controlled by turning the adjustable wheel at the cap.
3. This model has exchangeable front covers/caps.
4. This model has a Macro Zoom function: Working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

EarScope models

Intended use

The EarScope is intended to make images of the inner ear as part of an ear diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. This model has exchangeable front speculas. The speculas are provided in 3 sizes to be used for different sizes of the ear canal. Additional speculas are available for purchase from your regular supplier or from your Dino-Lite Medical reseller.
3. The pneumatic Earscope model has a bulb to blow air into the ear to check the mobility of the eardrum, which is sealed off by the speculum.

Note: Be careful not to apply too much pressure when squeezing the bulb.

CapillaryScope models

Intended use

The CapillaryScope is intended to make images of microcapillary vessels.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. The front cap has a special form to allow you to put it directly onto the rim of a nail.

How to use the CapillaryScope

1. To achieve the best results, it is advised to apply some vegetal oil (like peanut oil) to the area that is going to be examined (usually the ring finger).
2. Do not touch the oil surface before applying the Dino-Lite CapillaryScope.
3. Put the CapillaryScope right on the nail edge with the oil.
4. Adjust the focus and move the CapillaryScope slowly to find the edge of the nail. Then move the CapillaryScope to find the first row of the capillary vessel beneath the nail edge.

IriScope models

Intended use

The IriScope is intended to make images of the eye for eye diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. The IriScope has two light colors, white and yellow. The light color can be chosen from the software, white light is more suitable for light iris colors and the yellow light for darker iris colors.

 **Warning** note: Do not shine onto subjects eye for more than 4 minutes per day.

TrichoScope models

Intended use

The TrichoScope is intended to make images of hair and scalp as part of trichological diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. Some models (MEDL4HM and MEDL4HM4) have a polarization function that can be controlled by turning the adjustable cap.

PodoScope models

Intended use

The PodoScope is intended to make images of feet, toe nails and spots of the skin as part of a podiatric diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. This model has a polarization function that can be controlled by turning the adjustable ring at the cap.
3. This model has exchangeable front covers/caps.
4. This model has a Macro Zoom function: Working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

Generic models

Intended use

Generic Dino-Lite models can be used in general microscopy applications to assist in medical diagnosis.

Hardware features

1. Your Dino-Lite has a Microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. Models with the letter W in the article code have the Macro Zoom function: Working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

Software installation

The DinoCapture and DinoXcope software is licensed from Anmo Electronics Corporation and is subject to an End User License Agreement (EULA) that users will have to accept during the installation process.

 Important notice: DO NOT connect the USB cable of the Dino-Lite to the PC before installing the software.

1. Use the CD delivered with your Dino-Lite to install the DinoCapture and DinoXcope software and drivers. Alternatively, download and run the latest version of the software from the support section of the website: www.dino-lite.eu.
2. Click 'Next' and the Installshield wizard will start. (An 'Open File-security warning' may appear on some systems. Select 'Run' or 'YES'). Choose the language you want for the DinoCapture 2.0 interface.
3. Read the User License Agreement. If you agree, press 'Yes' to continue or press 'no' to stop the installation.
4. Select a destination folder for the DinoCapture software. When done, press 'Next'. Click 'Install' to start installing the software. If the Windows security warning message appears, click 'Install this driver software anyway'.
5. When the installation is complete, selecting 'Finish' completes the software installation.
6. The DinoCapture software has an auto-update feature that will check for software updates when you start DinoCapture.
7. A full manual can be found in the help function of DinoCapture,

as a pdf on the CD or on the website.

Hardware installation

1. After full installation of the DinoCapture software and driver package, connect the Dino-Lite to one of the USB ports of your computer.
2. Please use a USB 2.0 port that is fully powered. Some USB ports on portable computers do not supply sufficient power.
3. The driver will be automatically installed. Please WAIT until the notification appears: 'Device driver software installed successfully'.
4. Now start DinoCapture 2.0 by double clicking on the desktop icon.
5. The LED lights should go on and an image should appear in DinoCapture. If this is not the case, refer to the frequently-asked-questions (FAQ) on www.dino-lite.eu.

Hardware features

1. At the center of the device, the adjustable dial is used to set the focus. The focus of the image depends on the distance to the object. Once you have focused on the object, you can read the magnification rate achieved from the number next to the Δ symbol.

Software interface DinoCapture / Windows

There are five key sections in the DinoCapture software:

1. Menu bar
2. Image list tools
3. Preview window management bar
4. Tool bar
5. Preview window tools

Menu bar

	Folder – create folders or access existing folders.
	Files – manage files, such as open, copy, save, print or delete as well as send as e-mail or create slide show.

	<p>Settings – customize the way the software operates such as:</p> <ul style="list-style-type: none"> • Showing or hiding elements on the screen • The properties for measurements • The way the Microtouch or foot pedal (if attached) work • Accessing the Motorized focus accessory settings • Connecting over IP • Bar code recognition function • External GPS • Auto update function
	<p>Help – access the full user guide on screen, or read DinoCapture 2.0 information or the license agreement.</p>

Image list tools

	<p>Open – open a picture selected from the image thumbnail gallery below the image list tool bar.</p>
	<p>Copy – copy a selected picture to the windows clipboard.</p>
	<p>Save as – save selected picture(s) to the desired file format or folder.</p>
	<p>E-mail – attach the selected picture(s) to a new message opened in your default e-mail client.</p>
	<p>Print – print the selected picture.</p>
	<p>Slideshow – run a slideshow of the selected pictures.</p>
	<p>Delete – clean up the image thumbnail gallery by deleting the selected picture(s).</p>

Preview window management bar

This window enables you to switch between windows if two or more Dino-Lite's are connected at the same time or multiple pictures are opened.

Tool bar

The tool bar is divided into four parts:

Draw tools – The draw tool set allows you to write text and draw on the images. The bar is located below the preview window management bar.

Measurement tools – Many different measurement types can be done with the measurement tools. The double flash icon gives you access to the grid and ruler settings. To make a measurement on your picture, choose one of the measurement icons. Then read the magnification factor from the focus ring and input this value into the box to the right of the icon bar. For increased accuracy, perform a calibration with the aid of the calibration sheet that is supplied with your Dino-Lite.

⚠ Note: The values reached with the DinoCapture software are not intended to be used for diagnostic purposes.

Line	Left click and drag to the desired length, and click again to finish.
Continuous line	Click and drag to form one section of distance, click again to start another section. Continue until the total desired distance is measured. Double click to finish.
Point to line	Create a line to represent the base by clicking once to start, drag, and then click again to set the endpoint. Branch off from the base line to start measuring the line that is 90 degrees from the base line and a final click to finish the measurement.
Polygon	Click and drag to form the desired length, and click again to start the next section. When finished, double click to finish the polygon measurement.
Radius circle	Click and extend out to the desired radius.
Diameter	Click and extend out to the desired diameter.
Three point circle	Click on any three points on the circle you wish to measure.
Three point arc	Click on three consecutive points on an arc to measure.
Three point angle	Start at the pivot point and extend out to start measuring an angle.
Four points angle	Select two points from one line segment and another two points from another line segment to complete the angle measurement.
Center distance	After drawing at least two circles, select the icon and the mouse pointer will change to a pointing finger for selectable circles. Click on each circle and the software will measure the distance.
Gridlines	The pitch will match the magnification inputted.
Circle grid	The pitch will match the magnification inputted.

Crosshair on camera	The crosshair XY position can be moved when you select it. The cursor position is the location of the mouse pointer and location 0,0 is the center coordinates of the crosshair.
Scale crosshair	The scale crosshair increments can be compared to the scale on the bottom left corner. The centre of the crosshair can be moved. (Magnification value needs to be entered for this feature to work).

Text and line tools

Line format	Select line color, style and width
Font	Select font, font style, size and color

Measurement options

Measurement properties	Organize and show the status of all the measurements as well as the ability to adjust how the results are shown on the image.
Magnifier	Digitally enlarge the area around your mouse pointer for more details and accurate measurements.

Magnification	Input the magnification in the blue box. The magnification can be read from the dial on the microscope.
Units	Select the unit of measurement, inch, mil, mm or um.
Calibration menu	Select calibration, create new calibration profile or open the calibration folder.
Barcode Reader	Click on the icon to enable and disable barcode detection.

Preview window tools

	MicroTouch - this is activated when you touch the MicroTouch sensor. Activate or deactivate the MicroTouch. Some Dino-Lite models do not have this feature.
	Auto Exposure – This allows you to change the exposure or turn off auto exposure. Sliding the bar to the right increases the exposure time and vice versa.
	White Balance - Select the required White Balance Mode (available on certain models only).
	LED control - lets you turn ON/OFF or switch LED's on the Dino-Lite (most models).
	Setting - The setting window allows you to control the camera settings.
	Maximize - Lets you see the live video or picture in full screen. To exit full screen mode, click anywhere on the screen or press "ESC".
	Close window - Allows you to close the current window.

Dino-Lite Medical User Manual

	Snapshot - Take a picture.
	Video recorder - Start/stop recording of a video.
	Time-Lapsed Video - Create a series of images or videos taken at a regular time interval.
1280x1024 ▾	Choose the resolution for the image. Please note that an additional Codec may be required to display 5 megapixel resolution. See the support section on www.dino-lite.eu for further information.
Map: Default ▾	Choose the folder where images/videos will be stored
<i>Enter annotation here</i>	Click on the grey bar on the bottom of the preview window and the annotation box will expand. Write text within this box and press ENTER to start a new line. When done, left click anywhere in the preview window to save the annotation and exit the annotation box.
	Focus strength indicator - Indicates the level of focus achieved (available on certain models only)

Software interface DinoXcope / Mac

There are four main sections in the DinoXcope program window:


1. Action bar


- Make picture. If you want to make measurements on the picture later, open the picture by double clicking and add the input magnification that you read from the focus dial.
- Make video. Choose frame rate, recording time, quality and compression. To stop recording manually, use the ESC key

- Make time lapse video. Choose capture interval, number of frames and playback frame rate, as well as quality and compression.
- Switch LED's on/off
- Go to full screen mode, to end full screen mode use the ESC key

2. Image list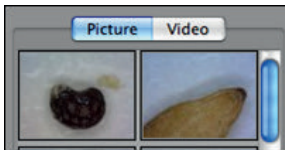

- Choose between pictures or video's
 - Look at an image with mouse-over, or double-click to open in new window
- ## 3. Live image
- Use the action bar for the action that you want to perform
 - Use the Controls menu (the menu bar is on top of the screen) to change settings, use (digital) zoom, freeze the image, change the resolution or change the functioning of the Microtouch button.
 - Use the Live measurement function (in the Tools menu) to open a special selection window for drawing and measurement tools. After selection, click okay and use the tool on the live image.
- ## 4. Open image

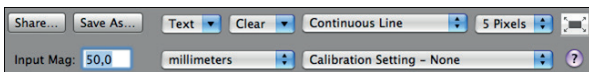- If you open a stored image, you will see a menu bar above it, that allows you to choose saving, drawing and measurement tools.

Note: The values reached with the DinoXcope software are not intended to be used for diagnostic purposes.

Cleaning and maintenance

Avoid touching the lens with any substance. Clean the lens periodically with compressed air. Clean the body of the microscope regularly with a soft cloth dampened with disinfectant. Clean the lens with 50%-70% alcohol solution, do not use diethyl alcohol gel.

Dispose as electronic waste according to local regulations.

Warranty

The DinoCapture software is provided for use with a Dino-Lite digital microscope supplied by AnMo Electronics or one of its distributors or resellers. The use of the DinoCapture software is guided by the End User License Agreement.

The Dino-Lite product is delivered with a two year warranty from the date of purchase by the end customer. Please note that accessories (i.e. detachable caps/extensions) are not covered under warranty. For warranty issues please contact the reseller or store where you bought the product.

Support

If you have any problem or issue with your Dino-Lite or the DinoCapture software, please contact your reseller or check the Dino-Lite Europe support pages at www.dino-lite.eu. An extensive user manual for DinoCapture can be found within the software and on the website.

IDCP B.V./Dino-Lite Europe
Manuscriptstraat 12-14, 1321NN Almere, The Netherlands
Tel. +31 20 6186322
Fax. +31 20 6189692
e-mail: info@dino-lite.eu
web: www.dino-lite.eu/medical

2021Q1 © AnMo/IDCP/Dino-Lite Europe.
No part of this publication may be reproduced or distributed unless for personal use in conjunction with usage of Dino-Lite digital microscopes.

Patent info:
US Pat. No. 7.102.817.B1
Germany Pat. No. 20 2006 001 409.8
Japan Pat. No. 3123176
Taiwan Pat. No. M302031

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Copyright 2021, IDCP BV. Alle rechten voorbehouden. Niets uit deze uitgave mag geheel danwel gedeeltelijk worden gekopieerd, vermenigvuldigd, gereproduceerd, of verspreid zonder uitdrukkelijke geschreven toestemming van IDCP BV.